



Strumen™ Tri Acid

Hot Sterilant Concentrate



Citric Acid based decalcifier
and sterilant Liquid

ACTIVE HYGIENE

Strumen™ Tri Acid

Hot Sterilant Concentrate

FOR EXTERNAL USE ONLY

Composition:

Each 100gms contains
Citric Acid I.P.: 21% w/v
Malic Acid and Lactic Acid as adjuvants
Purified Water I.P.: q.s.

Overview:

- Strumen Tri Acid-Hot Sterilant is used for Chemo-thermal disinfection of Hemodialysis machines with recirculation. It can be used in any Dialysis machine.
- It is used for descaling & decalcification of hemodialysis machines.

Features:

- Highly effective
- Dissolution of blood residues
- Excellent removal of CaCO₃
- Disinfection and decalcification in one process
- Active ingredients composed of natural ingredients: Citric acid, malic acid and lactic acid
- Strumen Tri Acid at 84°C has a broad spectrum of microbicidal activity and works bactericidal, virus inactivating (HBV, HCV, HIV) and fungicidal
- It replaces sodium hypochlorite and formaldehyde in terms of efficacy and safety.
- Nontoxic, biodegradable and environment friendly
- Compatible with all materials
- Transparent and Odourless liquid
- Free from colouring additives

Antimicrobial Activity :

- Destruction of the phospholipid layers in the cell membrane; disturbance of the intracellular pH balance
- Activity strongly increased by an increase in temperature.
- It is Bactericidal, Virucidal, Sporocidal and decalcifying agent.
- It kills bacteria, viruses and spores in short duration of time (5 to 10 Minutes).
- It is highly effective against hepatitis A, B viruses.

Directions for use:

- Use undiluted in Hemodialysis machine.
- Machine sucks predefined volume of Tri-Acid automatically during disinfection/sterilization cycle.
- Machine dilutes Tri-Acid with water.

Contact Time:	
For Disinfection	5 minutes
For Sterilization	10 minutes


Package: 5 Ltr Canister

Precautions/Contraindications:

- Wear safety goggles & gloves while handling.
- Wash immediately on contact with eyes or skin and seek medical help.
- Do not swallow.

Storage:

- Keep container tightly sealed in an upright position at room temperature

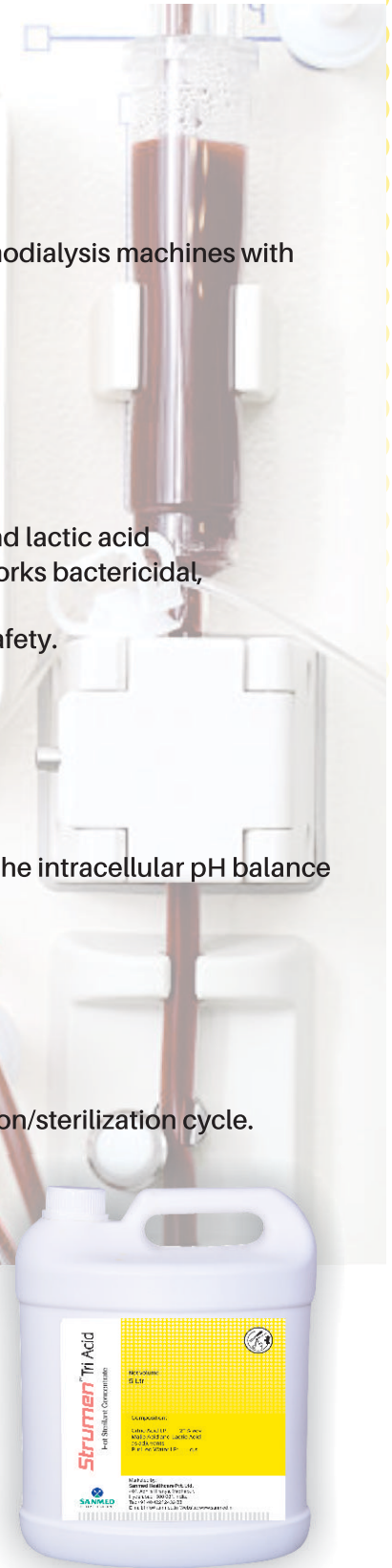

Our Accreditations:



Working towards actualization of



For more details, please contact:

Sanmed Healthcare Pvt Ltd
401, Abhi s Hiranya, Guttala Begumpet,
Madhapur, Hyderabad 500081
Telephone: 040 4221 2432

Email: info@sanmed.in Web: www.sanmed.in